

ETI

DENTAL INDUSTRIES LTD.


New Tel. For Factory and show Room 972-3-5799760, 972-3-5799759

16, HAKISHON Bney-Brak 51203 ISRAEL

Mail Address P.O.BOX: 1400 Bney-Brak 51114 Israel.


FAX. No. 972-3-5799758

CONNECTION FOR LIGHT ACTIVITY

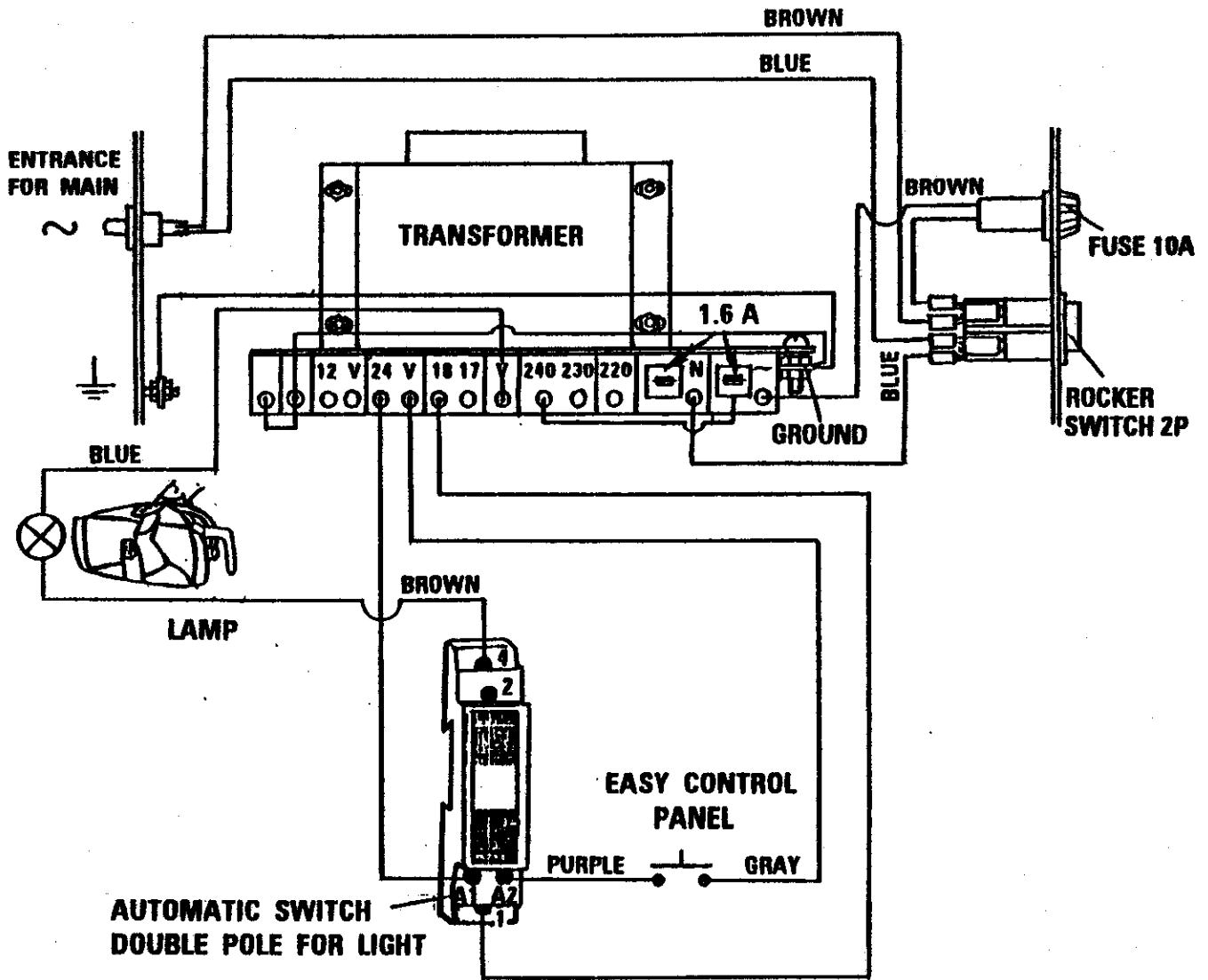

to CN2
**PANDA SCHEMA - HOW TO CONNECT CHAIR TECNODENT 5P - 2000
 TO HAND CONTROL 2P - 1000**



ETI COLOR WIRE

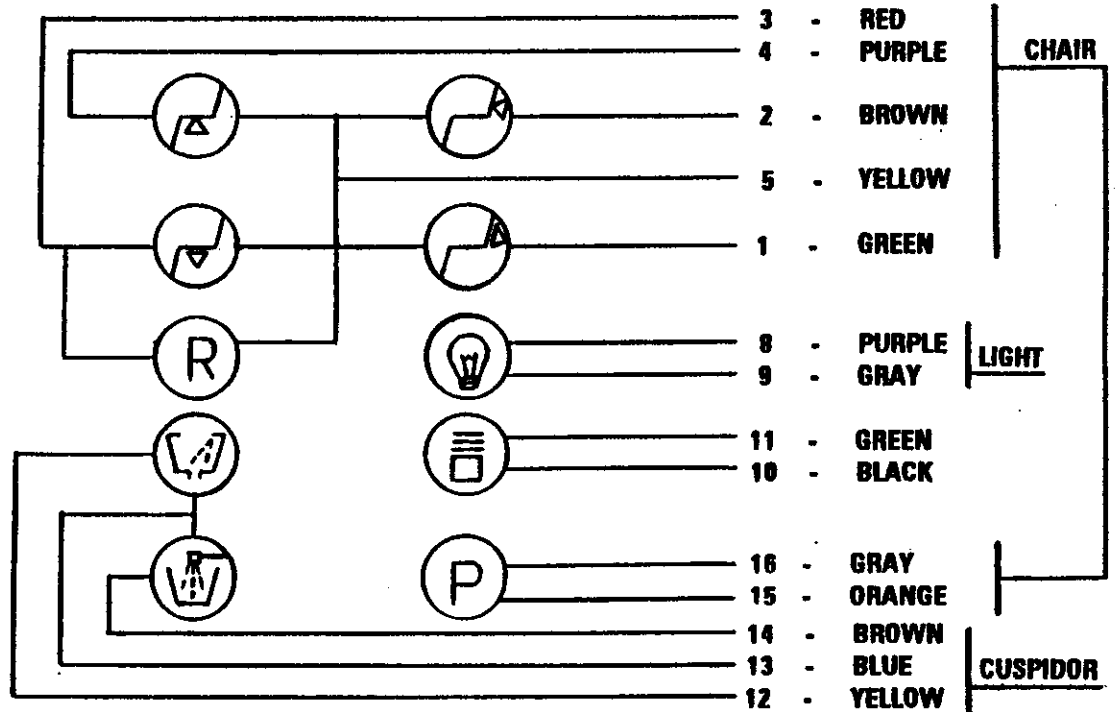

****CONNECTORS FROM TOUCH CONTROL TO UNIT PANDA PORCELAIN**


TO CHAIR:

TO : CUSPIDOR - 6, 7, 8. LIGHT - 1, 2 . TIMER (OPTION) - 3, 4 .

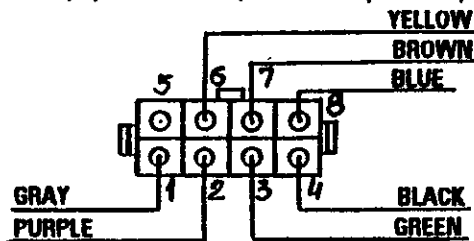
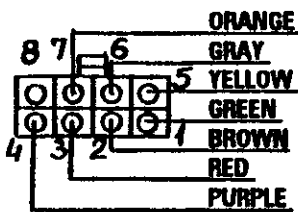



****CONNECTORS FROM UNIT PANDA PORCELAIN TO JUNCTION BOX**

TO CHAIR:

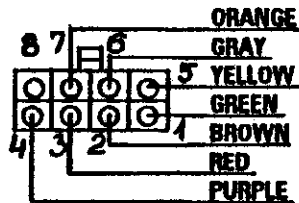

****CONNECTION TO CHAIR
 TECNODENT 5P - 2000 , 2P - 1000
 PLEASE CONNECT TO CN2 ONLY !**

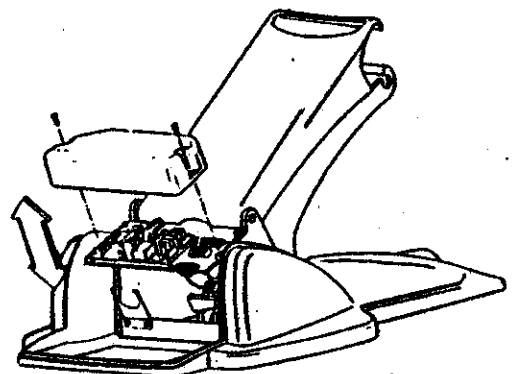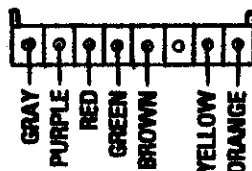

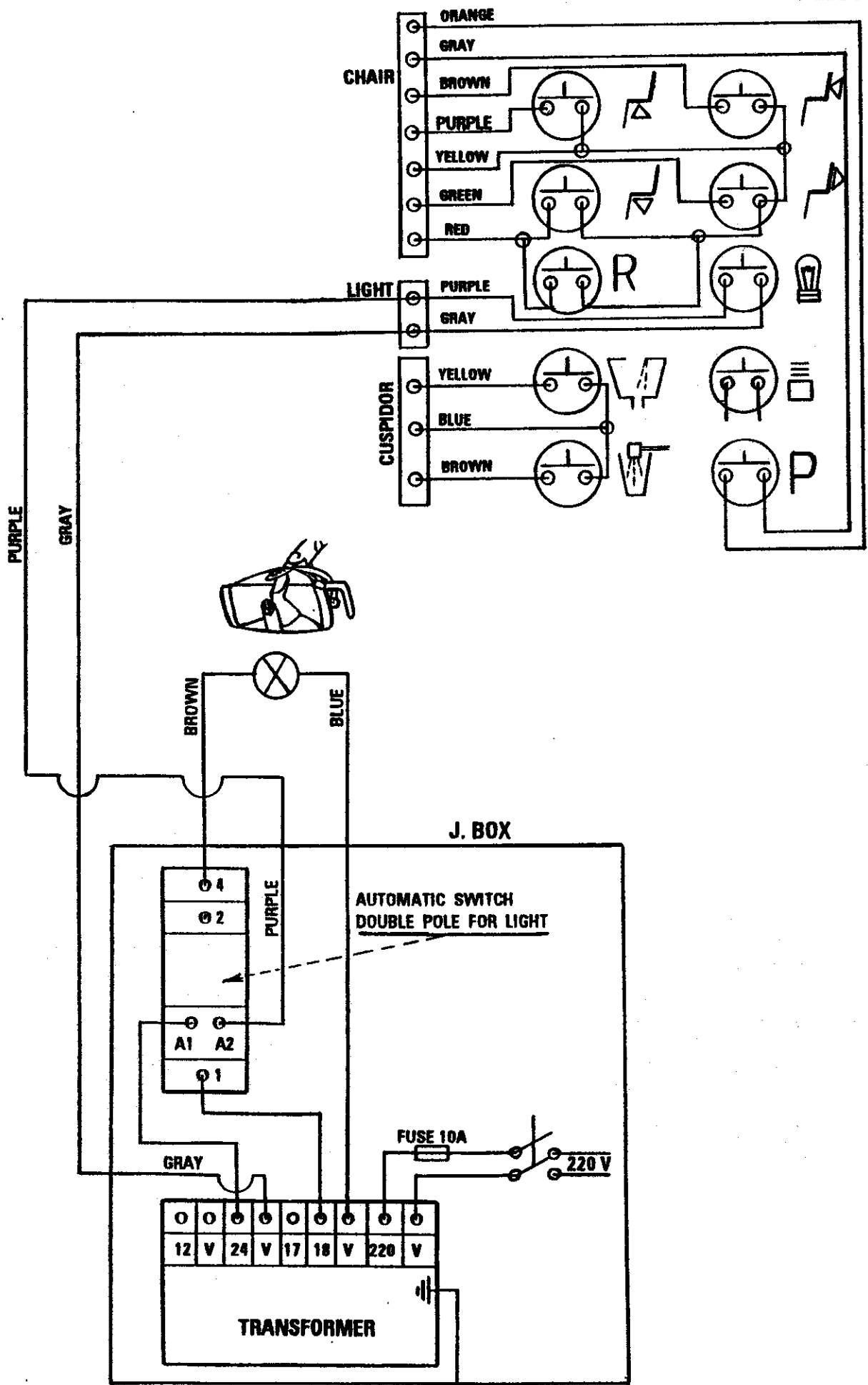

DRAW 1
2000

ELECTRIC SCHEMA FOR PANDA, GALAXY.
CHAIR WITH 5P, WITH 2P

ETI